

DISPOSABLE LAMINATED PROTECTIVE OVERALL

USER GUIDE



How to Wear

Make sure the size is compatible with the user. Do not make any changes on the product.

Check whether the product is flawless and in good condition (no holes, seamless parts, etc.)

Open the zipper and dress, taking care not to pierce the material. Close the zipper and cover the lid.

It is recommended to cover the zipper in the form of solid particles in the air and encircle the limbs and leggings with adhesive tape on the arms.

Protection Features apply when the disposable overall is properly worn and closed.

Storage

It is recommended to use the product within a period of five years from the date of manufacture printed on the label.

Warnings

Choose products that are compatible with the work area.

The disposable product must be changed after each use,

If there is any tear, punctures etc leave the work area and wear new overalls.



Prolonged wearing of protective clothing can cause heat stress. Heat stress and discomfort can be reduced or eliminated by using appropriate underwear or appropriate ventilation equipment.

This coverall meets the requirement of $15\% \leq Ls$ 8/10 $30\% \leq Ljmn$, 82/90

These garments are flammable - keep away from fire

If the product is damaged, leave the workplace immediately

The user should never remove the protective suit while using it in the Risk Zone.

Transport, Protection and Disposal

Disposable coverall should be stored in a dry place away from light and heat sources. If the product is not Contaminated, the product can be treated as normal textile waste. If contaminated, it is considered harmful garbage and must be disposed of according to country law.